




How to Place the AMT Bridle™ Nasal Tube Retaining System

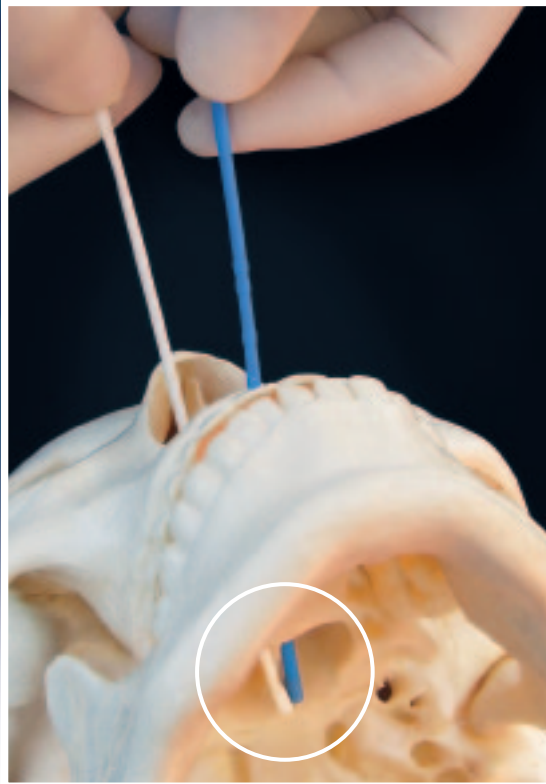


**Avoid Pullout Costs...
While Improving Patient Outcomes.**

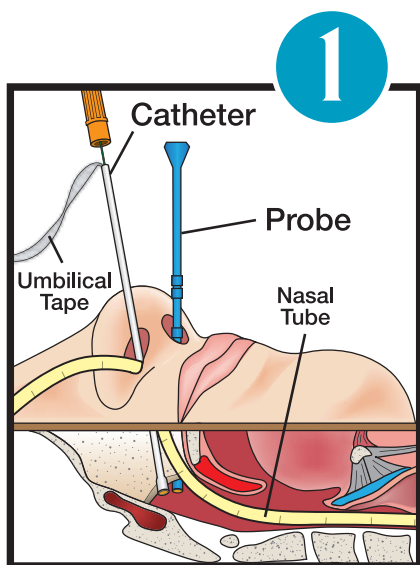


- Thoroughly lubricate catheter, probe and umbilical tape
 - Insert the blue probe and the white catheter into opposing nostrils, downward towards the floor of the nose
 - Gently manipulate until magnetic "click" is felt behind vomer bone

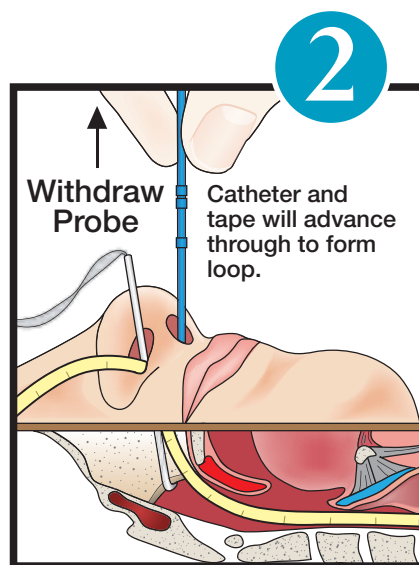

- Remove orange stylet from white catheter
 - Slowly withdraw blue probe and allow white catheter to follow
 - Cut white catheter, leaving only umbilical tape in place

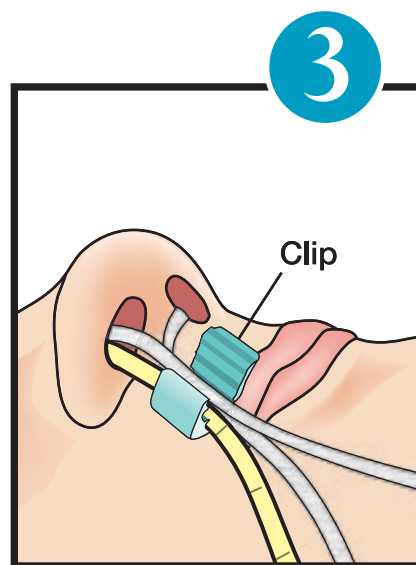

- Place nasal tube into clip channel
 - Place both strands of umbilical tape into clip
 - Position clip close to the nose

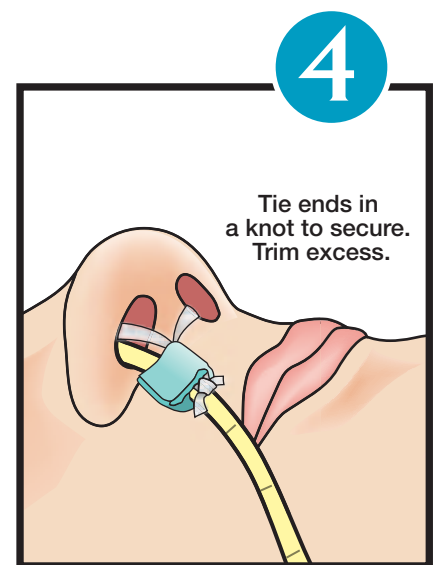

- Firmly close retaining clip until it snaps shut
 - Tie remaining ends of umbilical tape to secure
 - Trim excess

Removal of Device

Cut one strand of umbilical tape. Gently pull both the Bridle and nasal tube out of the nose.

Document removal in the medical record.

Clip Sizes and Re-Order Numbers

Clip Color	Clip Size	Bridle Kit Order Number	Spare Clip Order Number
Yellow	5F	4-4105	4-4505
Purple	6F	4-4106	4-4506
White	8F	4-4108	4-4508
Teal	10F	4-4110	4-4510
Blue	12F	4-4112	4-4512
Yellow	14F	4-4114	4-4514
Yellow	16F	4-4116	4-4516
Blue	18F	4-4118	4-4518
		5 units per box	5 clips per box

Indications for Use

- Prevention of nasal tube pullouts, accidental dislodgements and subsequent reinsertions

Contraindications for Use

- Patients with nasal airway obstructions & abnormalities
- Facial and/or anterior cranial fractures
- Basilar skull fractures
- Patients that may pull on the device to such a degree as to cause serious injury



The Leader in Enteral Device Innovation

An ISO 13485 Registered Company

8000 Katherine Boulevard
Brecksville, OH 44141

Toll-Free: 800-869-7382

E-mail: info@appliedmedical.net

www.appliedmedical.net



C-4328-B 09/2009



Watch the video at www.appliedmedical.net/amtbridle.htm